

GuttaCut®



Cordless device with interchangeable inserts for fast, precise and safe cutting of excess guttapercha points.



- The loop on the cutting tip provides easy access to the cavity and catches the excess guttapercha exactly at the canal orifice and precisely cuts it off.
 - The instrument is inserted cold and by pressing the button the tip of the insert heats up at an instant. When button is released, the tip cools down within seconds.
 - The hot cutting avoids an unintentional loosening of the filling material.
 - Gutta Cut® is always at your service, no preparation time or pre-heating required
 - Up to 80 cuts per battery charge. Fast recharging of the Ni-Cd power battery in the basic station.
 - Chairside, only the cordless hand piece is required, not the charging unit.
- Various instrument tips (inserts) are available.
- They are autoclavable for repeated use.



VW000098 Rev. 1/30.9.02

GuttaCut®

comfortable and time-saving



- 1 Insert cold instrument. Tip can be curved with tweezers.



- 2 Heat up and cut off all excess guttapercha at the canal access
- 3 Release button and withdraw the cold instrument with clinging guttapercha.

GuttaCut® program:



Ref. V040143
Gutta Cut® with charging unit
and 4 cutting tips



Ref. V040512
set of 4 cutting
tips



- 4 Done.

Technical specification:

Gutta Cut 175 x 20mm, 45g
Charging unit 134 x 90 x 18mm, 145g
Rechargeable Ni-Cd battery 1,2V
Battery charger 220-240 V, 50-60 Hz / 2,5W
Cable length 140mm
Each device comes with a user manual